



Quick Start Guide - PLEXTALK PTN2

Description

The PLEXTALK is a portable, handheld talking book player.

To Use

The PLEXTALK PTN2 comes with an easy operation cover. The PLEXTALK can be operated with or without the cover. First, instructions will be presented while using the cover, followed by instructions for using PLEXTALK without the cover.

How it is organized – PLEXTALK with Cover

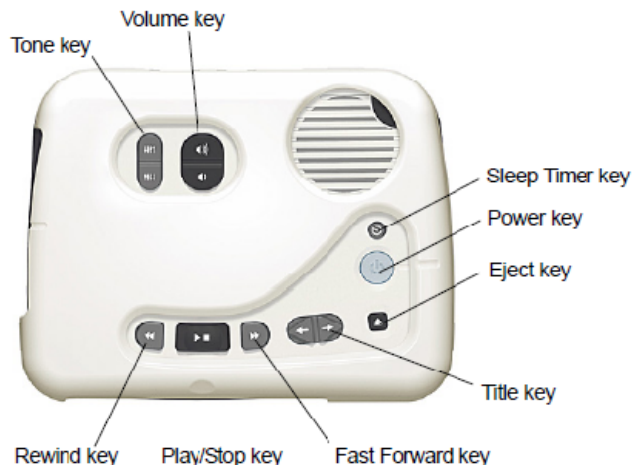










Figure 1. Front of PLEXTALK with Easy Operation Cover Installed

- Speaker is at the upper right on the top surface.
- The sleep timer, power, eject, and title key are under the speaker.
- The tone and volume keys are on the top, to the left of the speaker.

- The rewind, play/stop, and fast forward keys are on the bottom.
- The DC input terminal is on the back.
- The SD card slot, headphone jack, and USB slot are on the left side edge.
- The CD slot is at the front side.

Function of the Keys

- Power Key  : Hold this key for 2 seconds to turn PLEXTALK on and off.
- Sleep Timer Key  : Used to set the time for shutting down. Press it to set the sleep timer for intervals between 15 minutes to 1 hour.
- Eject Key  : Used to eject a CD.
- Title Key  : To go to the previous title, press the left arrow. To go to the next title, press the right arrow. To select CD, SD, or USB, press both the left and right arrow sides of the title key together.
- Tone Key  : To increase the tremble or bass sound, press the top or bottom of this key.
- Volume Key  : To increase the volume, press the top of the key. To decrease the volume, press the bottom of the key.
- Rewind Key  : To move back to the previous phrase in a DAISY book, the next track in an audio file, or the next track in an audio file briefly press this key. To rewind fast press and hold this key.
- Play/Stop Key  : This key starts and stops playback.



- **Fast Forward Key** : To: move forward to the next phrase in a DAISY book, the next track in an audio file, or the next paragraph in a text file, briefly press this key. To fast forward, press and hold this key.

How to Play a DAISY book on a CD Using the PLEXTALK with Cover

1. Put the CD disc into the CD slot at the front side of the PLEXTALK.
2. To play the book, press the **Play/Stop Key**.
3. To stop the playback press the **Play/Stop Key**.
4. To forward the CD press and hold the **Fast Forward Key**.
5. To rewind the CD press and hold the **Rewind Key**.

How to play audio Using the PLEXTALK with Cover

1. To select the audio, press and hold the left of right arrow on the **Title Key** and then press the **Play/Stop Key**.
2. To select a track press the fast forward or **Rewind Key**.

How it is organized – PLEXTALK without the Cover

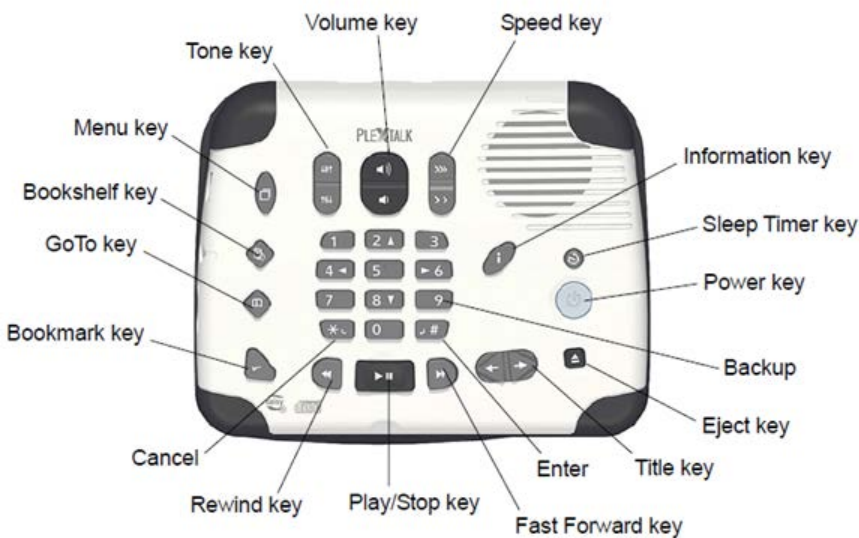




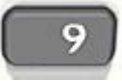





Figure 2. Front of PLEXTALK without the Easy Operation Cover

- Speaker is at the upper right on the top surface.
- The information, sleep timer, power, eject, and title key are under the speaker.

- The tone and volume keys are on the top, to the left of the speaker.
- The rewind, play/stop, and fast forward keys are on the bottom.
- The DC input terminal is on the back.
- The SD card slot, headphone jack, and USB slot are on the left side edge.
- The CD slot is at the front side.

Functions of Additional Keys

The function of the main keys was described in a previous section. Removing the cover exposes additional keys, whose functions are explained here.

- Information Key  : Information about the battery and current title
- Speed Key  : Increase or decrease the speed
- 9 Key  : Backing up a selected title on CD, SD, or USB.
- 0 Key  : Activates the help function which contains the built-in manual.
- Star Key  : Cancel entries in title selection, the bookmark, the menu option, etc.
- Hash Key  : Confirm entries in title selection, the bookmark, the menu option, etc.
- Menu Key  : Use the menu key to activate the following menu choices: playback settings, skippable, media management, and system settings press the menu key. To enable key describer mode, hold the menu key.
- Bookshelf Key  : Press the key repeatedly until the desired media is announced.



- GoTo Key : Navigate directly to a page or heading in a DAISY book, to a track or an album in audio or to a percentage in a text.



- Bookmark Key : To go to a bookmark press once. To create a new bookmark press twice. To remove a bookmark press three times.

How to Play a DAISY book on a CD Using the PLEXTALK without the Cover

1. Put the CD disc into the CD slot at the front side of the PLEXTALK.
2. To play the book, press the **Play/Start Key**.
3. To stop the playback, press the **Play/Start Key**.
4. To forward the CD, press and hold the **Fast Forward Key**.
5. To rewind the CD, press and hold the **Rewind Key**.
6. To select the navigation level, press the **2 or 8 Key**.
7. To move backward or forward in the selected level, press the **4 or 6 Key**.

Web Resources

User Guide for the POCKETALKER on the Plector website:

http://www.plector-europe.com/products/docs/pocket_guide/PTP1PocketGuideEnglish.pdf

Plector Contact Information

1078, Kami-maruko
Ueda-shi, Nagano-ken 386-0498
JAPAN

Email: service@plextor.be

<http://www.plector-europe.com/index.html>